GB

Inflation and Deflation Instructions for Foot Protector

Inflation

- 1 Remove the device from the tube and refit the two tubes together to form a pump.
- 2 Connect the valve on the pump to the valve on the device - Push fit.
- 3 Inflate the device until air is heard escaping from the pump valve.
- 4 Cover the valve on the device using the solid end of the 'figure of eight' valve cap.
- 5 Turn device over with valve away from patient.
- 6 Secure the Footprotector using a light bandage or stocking.

NOTE: Check device daily to ensure proper inflation level. In case of doubt, repeat pumping procedure above with the user off the product.

Deflation

- 7 Insert the open end of the 'figure of eight' valve cap into the device valve. Air can now escape from the valve allowing the device to deflate.
- 8 Roll up the device starting at the point furthest away from the valve. Continue until all the air is expelled.
- 9 Clean the device and allow to dry. Replace device into the pump.
- 10 Refit the two tubes together and rotate outer tube until it clicks closed.

Limitations for use

Repose is not suitable for the following:

- Repose is not suitable where the size or shape of the persons limb is such that it cannot be fully supported by the Repose Footprotector

Precautions

- If deterioration in skin condition is noted, clinical advice should be sought and if so advised, discontinue use of product
- Avoid direct contact with sharp objects
- When deflated, keep away from babies and small children
- Keep away from pets

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REPOSE CAN BE CLEANED AND REUSED

WIPE CLEAN DO NOT DRY CLEAN DO NOT TUMBLE DRY DO NOT IRON

REPOSE CAN BE CLEANED WITH SOAP OR DETERGENT AND WATER AND CAN BE DISINFECTED IN ACCORDANCE WITH THE INFECTION CONTROL POLICY FOR STANDARD MATTRESSES ALCOHOL & PHENOL BASED DISINFECTANTS SHOULD NOT BE USED ALLOW TO DRY BEFORE USE OR STORAGE

REPOSE IS SUPPLIED WITH A MANUFACTURERS WARRANTY FOR 12 MONTHS FROM DATE OF DISPATCH

IMPORTANT NOTICE
CHECK PRODUCT DAILY TO ENSURE CORRECT INFLATION LEVEL IN CASE OF DOUBT, REPEAT PUMPING PROCEDURES WITH THE PATIENT OFF THE PRODUCT

KEEP AWAY FROM DIRECT HEAT AND SHARP OBJECTS

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REPOSE WIRD MIT EINER HERSTELLERGARANTIE VON 12 MONATEN AB DEM VERSANDDATUM DELIEFERT.